


How to Log In

Visit myvirena.com using one of the following internet browsers:

- Chrome latest Version or above running on Windows 10
- Firefox
- Edge
- Internet Explorer version 11 or above

What to Expect

The first page is the dashboard which displays a comprehensive organization snapshot along with a side bar of navigational options. Note: the default filter applied to the displayed data is 90-days.

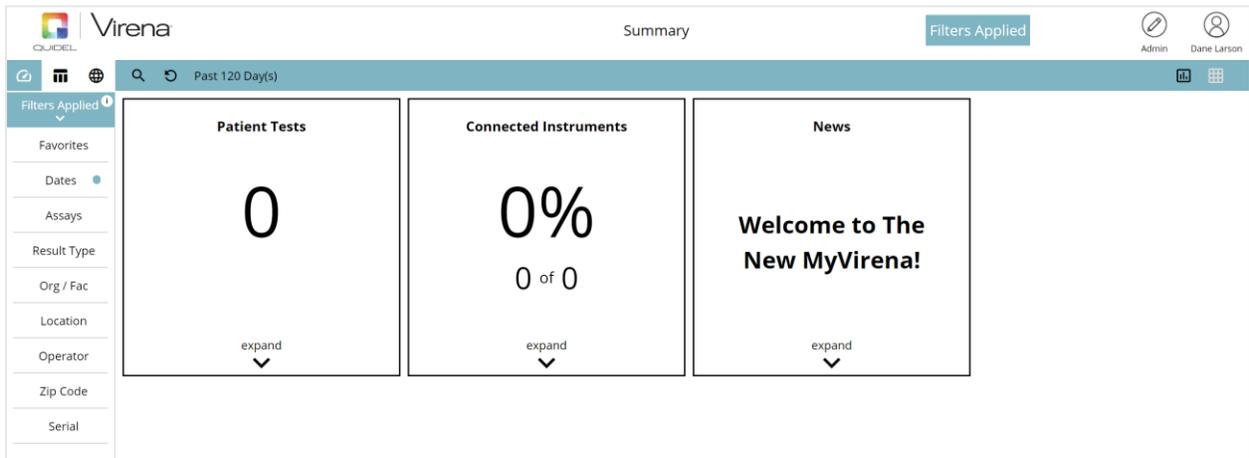

The screenshot shows the MyVirena dashboard interface. At the top, there is a header with the QUIDEL logo, the word "Virena", and the text "Summary". On the right side of the header, there is a "Filters Applied" button and user information for "Admin" and "Dane Larson". Below the header is a navigation bar with a search icon, a refresh icon, and the text "Past 120 Day(s)". On the left side, there is a sidebar with a "Filters Applied" dropdown menu and a list of filter categories: Favorites, Dates, Assays, Result Type, Org / Fac, Location, Operator, Zip Code, and Serial. The main content area is divided into three panels: "Patient Tests" showing "0", "Connected Instruments" showing "0%", and "News" showing "Welcome to The New MyVirena!". Each panel has an "expand" button at the bottom.

Navigation Icons

-  **Dashboard Summary:** This is where you can get an overview of patient results, connected instruments, and MyVirena news.
-  **Test Results:** All results will be displayed in this view. Selecting a specific result will provide the details of the test. Results can be filtered to generate reports.
-  **Mapping Tool:** Provides national view of all connected instruments. Results can be displayed by Assay or Organization. Selecting the specific facility or county push pin will show aggregated totals and percent positivity rates.
-  **Charting Test Results:** Generate test result charts by date, assay, volume and test type.
-  **Export Results:** Once a report has been created, a .csv file can be exported to your email by clicking this icon.

Filters

Apply one or more filters to generate a MyVirena Report. Applied filters will appear in the toolbar shown below:



Note: The filter reports by run date. MyVirena defaults to results run in the last 90 days.

Favorites

Save frequently used filter configurations. To save a configuration, apply the desired filters, select the favorite filter and click +. Favorite reports can be scheduled and automatically sent to authorized users.

Reports

Within Virena, you will have access to nine tailored reports that capture your organization's unique data. Examples of these reports include:

- Patients By Run Date
- Patients By Assay
- Patients By Facility/Assay
- Patients By Facility/Result
- Patients By Result
- Patient Trending
- Percent Positives
- QC Results
- Volumes By Type

Additional Resources

If you currently utilize the MyVirena.com, you will immediately notice these changes and can go to connectme.quidel.com to view training videos, request additional training, or simply discover more ways that Virena can add value to your testing algorithm.

For additional assistance, please contact Quidel Technical Support at 1.800.874.1517.

QR1011000EN00 (06/20)